

3 Quick Steps to Print Reports

Access all of your patients' cloud-based data from any computer at any time. Download both summaries and detailed reports for discussions with your patients about their management strategies.



1 Transfer data from the pump


- A** Open the t:connect Uploader software by double clicking the icon on your desktop or applications folder.



- B** Connect a Tandem pump or supported blood glucose meter to your computer using the appropriate USB cable.




- C** Click the **Start Upload** button.

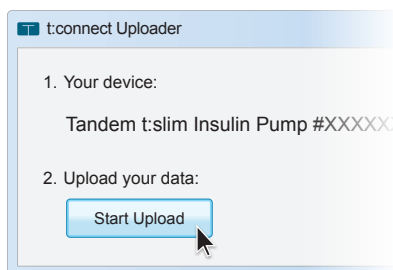


2 Sign in to the t:connect Application

- A** Once you see the Upload Successful message, click **View Reports**.

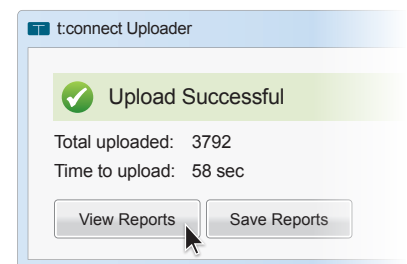

- B** Enter the email address and password that the patient used to create the account.

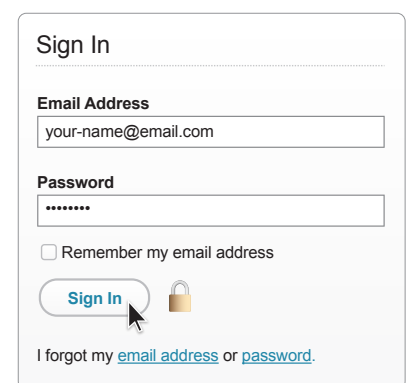

- C** Click the **Sign In** button.

3 Save and print reports

A Select the **SAVE & PRINT** page from the top menu.

B Check the box next to your preferred reports and select the desired time period for each one. Your selections may include the Dashboard, Therapy Timeline, Blood Glucose Trends, Logbook, Activity Summary, and Pump Settings.

Note: If the user has a t:slim G4 Pump, you can select to view either the CGM Logbook with the CGM Hourly Report or the Standard Logbook.

C Click the **Save & Print** button to generate the patient report.

Note: If desired, you can save the file to your computer for future reference or for upload to a compatible EMR system. Whether you save or not, all patient data is available in the t:connect Application and can be retrieved at any time.

touch simplicity®

TANDEM
DIABETES CARE

1-877-801-6901
www.tandemdiabetes.com

Connect with us on:



Important Safety Information

The t:slim G4 Insulin Pump is intended for the subcutaneous delivery of insulin, at set and variable rates, for the management of diabetes mellitus in persons requiring insulin, for individuals 12 years of age and greater. The t:slim G4 System also includes a continuous glucose monitoring device indicated for detecting trends and tracking patterns in persons with diabetes. The t:slim G4 Pump is for sale on or by the order of a physician. For detailed indications for use and safety information visit <http://tandemdiabetes.com/SafetyInfo/#t:slimG4safetyinfo>.

The t:connect Diabetes Management Application is intended for use by individuals with diabetes mellitus who use Tandem Diabetes Care insulin pumps, their caregivers, and their healthcare providers in home and clinical settings. The t:connect Application supports diabetes management through the display and analysis of information downloaded from Tandem Diabetes Care insulin pumps and specified blood glucose meters.

© 2016 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, t:connect, and touch simplicity are registered trademarks and t:slim G4 is a trademark of Tandem Diabetes Care, Inc. ML-000370_A