


View Status

for the t:slim X2 Insulin Pump




UNITS OF MEASURE MMOL/L

The instructions below are provided as a reference tool for caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. Not all screens are shown. For more detailed information on the operation of your Tandem insulin pump, please refer to its user guide.

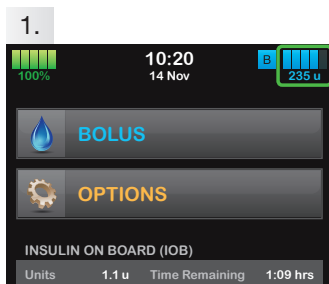


1. Tap the insulin level icon in the upper right corner of the Home Screen.

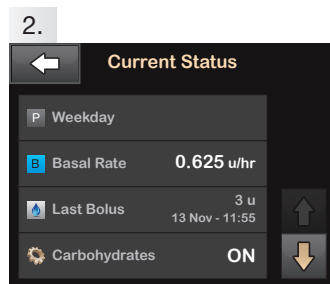


2. Your t:slim X2 pump will display the name of your active profile, your current basal rate, the time and amount of your last bolus, and whether or not you are entering boluses based on carbohydrates.

If you have an active Extended Bolus, this screen will show how much insulin has been delivered out of the total amount requested.

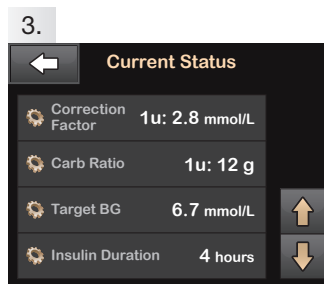


3. Tap the **Down Arrow** to display your current Correction Factor, Carb Ratio, Target BG, and Insulin Duration.

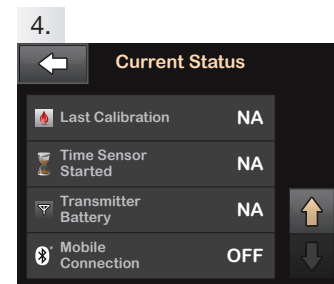

4. Tap the **Down Arrow** to display your Last Calibration, Time Sensor Started, Transmitter Battery status, and Mobile Connection status.

Explanation of Icons

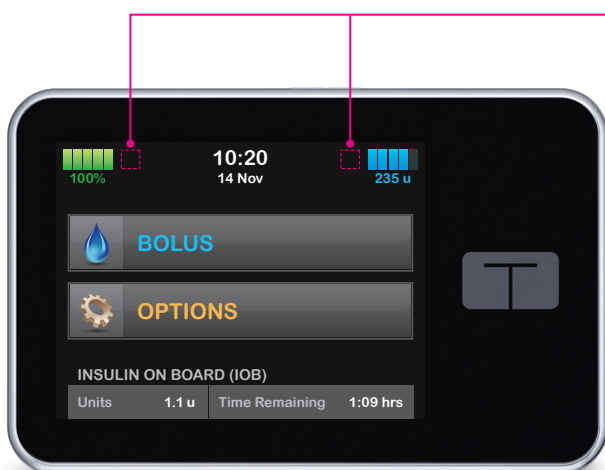

The following icons may appear in the areas to the left or right of the time and date on your t:slim X2 pump's Home Screen.



A system reminder, alert, error, or alarm is active. Unlock screen to view.



A bolus is being delivered.



All insulin deliveries are stopped.



Basal insulin is programmed and being delivered.



A basal rate of 0 u/hr is active.



A temporary basal rate is active.



A temporary basal rate of 0 u/hr is active.



(833) 509-3598
tandemdiabetes.ca



t:simulator App
A demo as simple as our pumps!



FOR USE IN CANADA ONLY

