

# Tandem Mobi™

Adhesive Sleeve



**TANDEM™**  
Diabetes Care



# Instructions for Use


The Tandem Mobi™ adhesive sleeve is a non-sterile accessory that may be used to secure the Tandem Mobi insulin pump for up to three days. The sleeve is intended for single use application, do not attempt to reuse or reapply the sleeve.

## **▲ WARNINGS**


- DO NOT use sleeve if you have sensitive, irritated, fragile, or easily damaged skin or if you have an allergy to one or more acrylics.
- DO NOT place the sleeve on mucous membranes or compromised skin.
- ALWAYS insert infusion set and connect to the pump before applying the sleeve to ensure the tubing is not stretched. Failure to follow these steps could result in crimping or dislodgement at the infusion site, which could affect the performance of the cannula. This could lead to hyperglycemia (high blood glucose)

## **▲ PRECAUTIONS**


DO NOT wear or place your pump more than 12 inches above your infusion site. Doing so may result in over delivery of insulin.

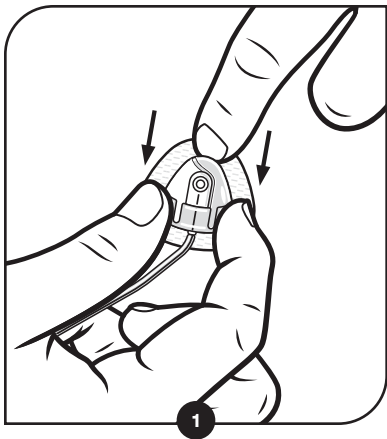

After completing the steps to load a cartridge using the Tandem Mobi mobile app, disconnect the tubing at the infusion site.

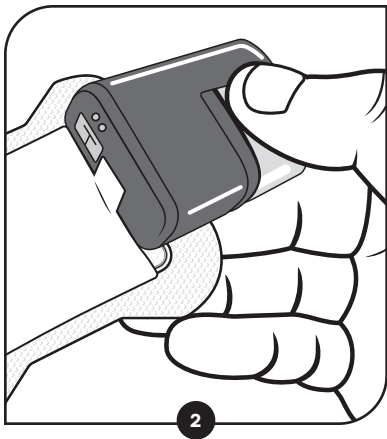


Place the pump with the Pump button touching the sleeve.

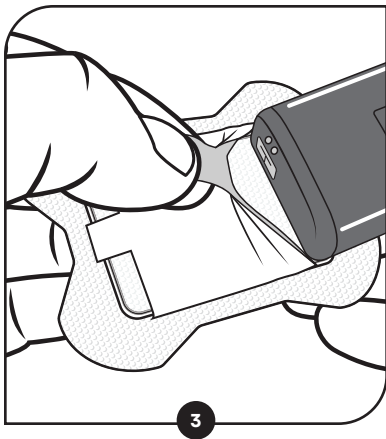


3

Lift and hold the tab at the end of the sleeve.

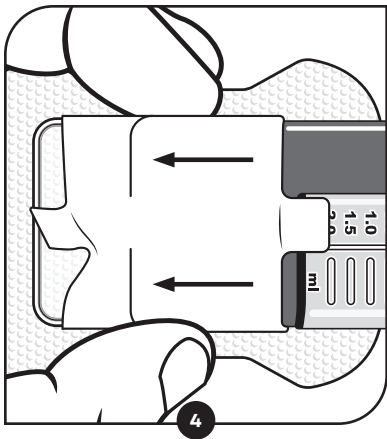


Push the pump through the sleeve until it is fully seated into the tray.

Continue these steps on the back of this sheet →

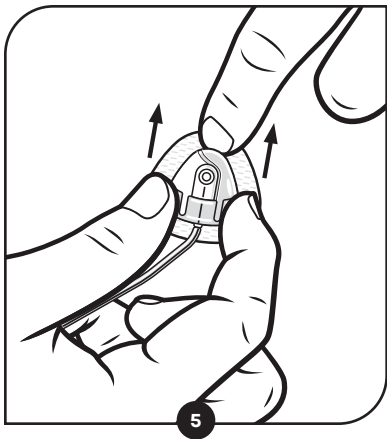


Connect the infusion set tubing to the infusion site.



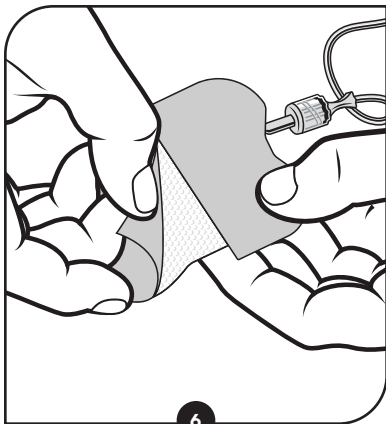


6

Grip the pump with one hand while you peel off the adhesive paper backing.

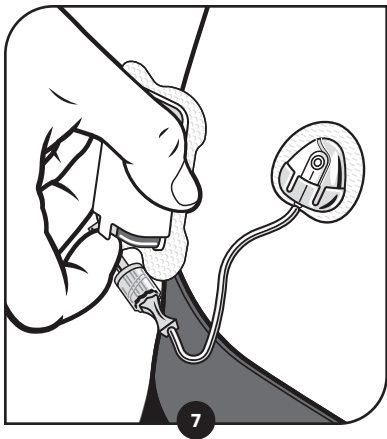

Apply the sleeve onto your skin  
at the chosen location

**Tip:** When placing the sleeve onto your skin, it's helpful to keep the pump button and LED Status Indicator facing up for easier visibility.

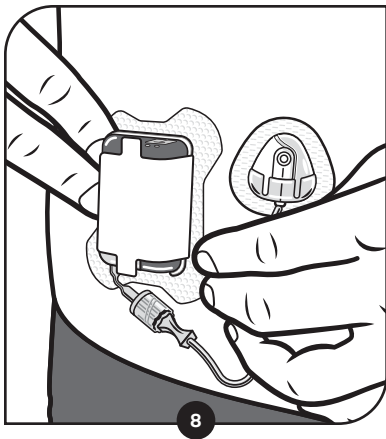















Smooth out the fabric and  
press around the edges.

## Location Placement Tip

- Ensure the location is clean and dry before applying the sleeve. Using lotions or moisturizers on skin prior to placement may affect adhesion.
- Assistance may be required when applying your sleeve in a hard-to-reach location.
- Always rotate the application site of the Adhesive Sleeve and avoid consecutive use of the same site.


## Explanation of Symbols

Symbol	Meaning
	Do Not Re-Use
	Consult Instructions for Use
	Humidity Limitation
	Temperature Limit

Symbol	Meaning
	Use by Date
	Manufacturer
	Date of Manufacture
	Country of Manufacture
	Catalogue Number
	Batch Code
	Manufacturer Number
	Instructions for Use
	Adhesive Sleeve
	Quantity



**TANDEM**  
Diabetes Care

 Tandem Diabetes Care, Inc.  
12400 High Bluff Drive  
San Diego, CA 92130 • USA  
+1-858-255-6269 • [tandemdiabetes.com](http://tandemdiabetes.com)

**2023-JUL-15**

© 2023 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, the Tandem logo, t:connect, and Tandem Mobi are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. All third-party marks are the property of their respective owners.

**1008667\_H**  
AW-1008669\_G